


# Single Leads ECG Module **MECG1**

The MECG1 single-lead ECG measurement module can measure ECG and is used in patient monitoring.

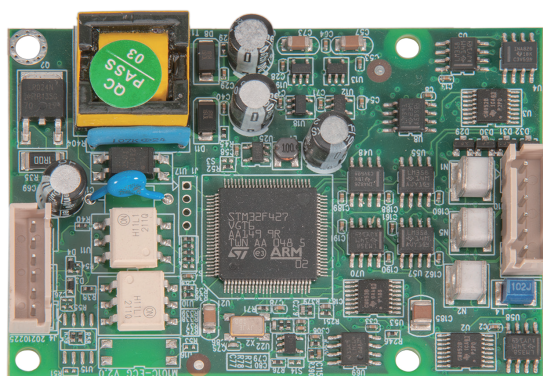

Dimension : 77 m m × 53 m m × 17 m m

## Features

- > With 3-lead ECG monitoring function
- > Patient model: adult, pediatric and newborn
- > 26 arrhythmia analysis functions, verified by MIT and AHA databases, and passed EC57 standardized tests
- > Provides four measurement modes: diagnosis, monitoring, HARDEST and surgery mode

Diagnosis mode: filter range is 0.05hz~130hz

Monitoring mode: filter range is 0.5hz~40hz

HARDEST mode: the filtering range is 5hz~20hz

Surgery mode: filter range is 1hz~25hz

- > ECG calibration: input 1mv standard voltage to record the waveform amplitude
- > Gain setting: adjust the amplitude of the ECG waveform
- > Notch mode: 50Hz, 60Hz, 50/60Hz and close notch mode commands can be selected

## Specifications

ECG		HR	
Range	0.15Mv-5.5mV	Range	15~300bpm
Accuracy	Undefined	Accuracy	±1bpm
Resolution	2.36uV/LSB	Resolution	1bpm
lead type	3 Lead:I or II or III		

## Electical Specification

Power supply	DC 5V±5%
Power consumption	≤3W
Communication	TTL,USART
Temperature	Operating 0°C~ 70°C Storage -40°C~ 70°C

## Compliance


Standard	IEC 60601-2-27: 2011 AAMI EC57:2012/(R)2020
----------	--

## ECG algorithm

Database	Sensitivity of QRS wave detection/ Q Se	Positive predictive degree of QRS complex detection/Q +P	Sensitivity of room early detection/ V Se	Early detection accuracy rate /V +P	False positive rate of early detection /V FPR
MIT	99.60%	99.72%	92.0%	92.5%	0.52%
AHA	99.66%	99.91%	90.0%	89.31%	0.638%

## Dimensions


The board dimensions are given in millimeters.

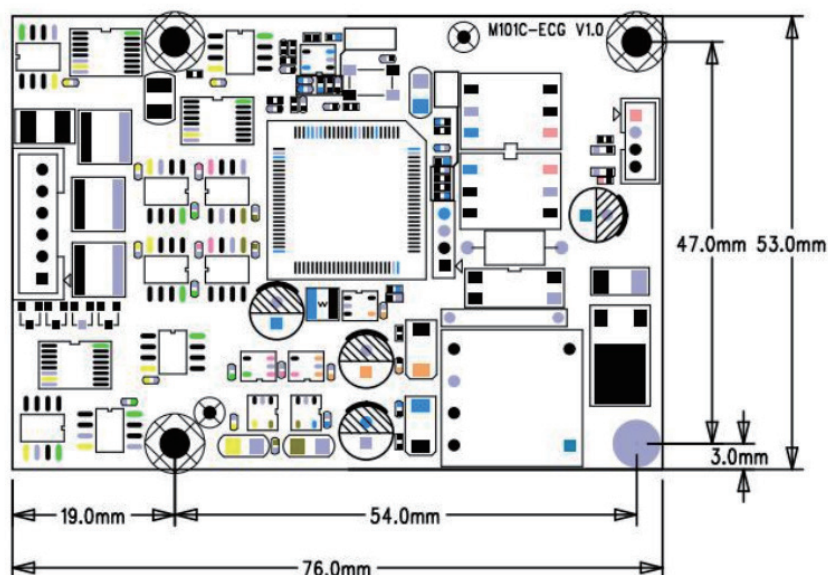

# Interfaces

## Power and Communication Interface

Note 1: TXD is the data sent by the ECG module to the host, in TTL level mode.

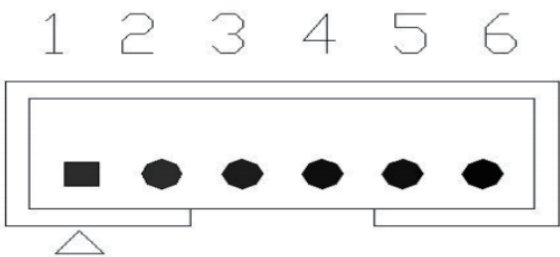


Power and communication interface socket J4 pins are defined as follows:


Pin No	Signal	Description
1	+5V	5V power input
2	GDN	5V power ground
3	RXD	UART Sending data from
4	TXD	UART Receiving data from
5	R Flag+	R-wave positive 5V pulse
6	R Flag+	R-wave negative 5V pulse

## ECG lead input interface

The lead input interface J10 is defined as follows, and the socket spacing is 2.54mm:

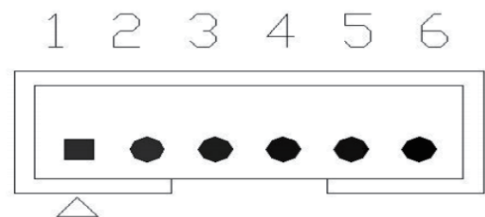



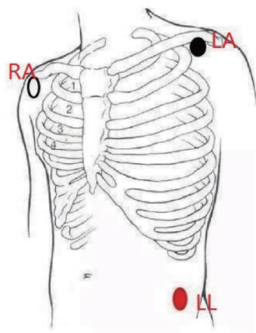


A schematic diagram of the location of the lead electrodes is shown below:



Pin No	Signal	Description
1	RA	Installed under the collarbone, near the right shoulder
2	LA	Installed under the collarbone, near the left shoulder
3	LL	placed in the left lower abdomen
4	V1	Chest leads in the fourth intercostal space, close to the right border of the sternum
5	ECG SHIELD	Signal shield wire
6	RL	placed in the lower right abdomen

## Purchase Guider

Code	Name	Quantity	Description	Image
022-060201-00	MECG1 Module	x1		
009-130101-00	Internal Cable ECG	x1	Connect with Module interface	
022-990101-00	Connector board	x1	Connect with Internal Cable ECG	
041-004001-02	<b>ECG Cable</b> 3-lead defibrillation proof, European standard, button type	Optional	Connect with connector board ECG interface	
041-004001-06	<b>ECG Cable</b> 3-lead defibrillation proof, American Standard, button type	Optional	Connect with connector board ECG interface	

\* The data is subject to change without notice. Please refer to the manual for the contraindications and precautions

Headquarter  
ZUG MEDICAL SYSTEMS SAS  
136 Chemin des Breguieres  
83170 Brignoles, France

Tel:+33(0)9 84 116 339  
sales@zugmed.com  
www.zugmed.com

**ZUG**<sup>®</sup>  
MEDICAL SYSTEMS